


## Step 1

Fold the middle of the nose clamp lightly. Expand pleats.

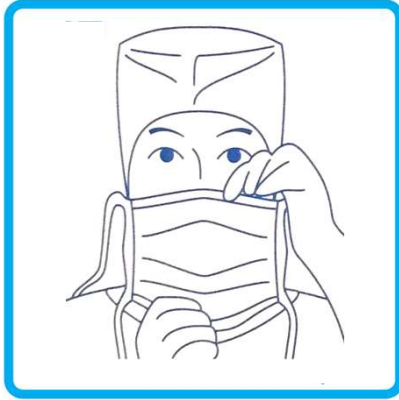


## Step 2

Conform nose clamp to your nose.




## Step 3

Tie the upper string firmly at the top of your head.

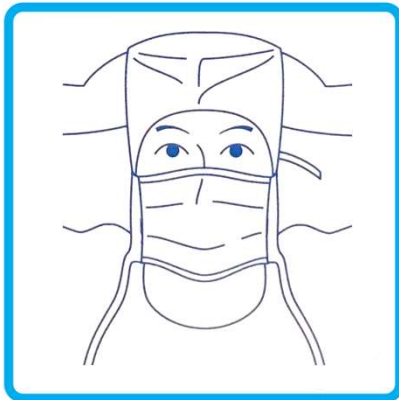

## Step 4

While holding down nose clamp, place the bottom of the mask under your chin.

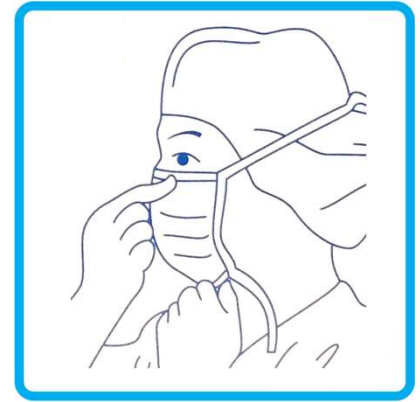

## Step 5

Adjust so there is no gap between both cheeks. Secure the lower strings at the neck.



## To Order

- Call (888) 976-8334
- Scan the QR Code on the right
- Email [Sales@Synergy-Healthcare-Solutions.com](mailto:Sales@Synergy-Healthcare-Solutions.com)
- Visit [www.Synergy-Healthcare-Solutions.com](http://www.Synergy-Healthcare-Solutions.com)



Distributed by



**SYNERGY**  
Healthcare Solutions

## Precautions for Use

- This product is a disposable mask. Do not wash and reuse.
- If you feel abnormal after wearing this product, stop using it immediately.
- This product is not an industrial dust and gas mask. Do not use the mask in environments generating harmful gas or dust.
- Be careful not to hurt your face or hands with the nosepiece.
- To maintain proper hygiene, replace the mask immediately when it gets dirty.

## Storage

Store at room temperature in a clean, dry place, away from direct sunlight. Keep the product out of reach of children.



## To Order

- Call (888) 976-8334
- Scan the QR Code on the right
- Email [Sales@Synergy-Healthcare-Solutions.com](mailto:Sales@Synergy-Healthcare-Solutions.com)
- Visit [www.Synergy-Healthcare-Solutions.com](http://www.Synergy-Healthcare-Solutions.com)



Distributed by



**SYNERGY**  
Healthcare Solutions